

## Instructions for Cloth Masks & Bags

Employees have all received a cloth mask, paper bags, and a clear freezer bag.

Take your clean mask out of the freezer bag to wear and keep the freezer bag for clean mask storage. Put your mask on.



### When wearing your mask please be mindful of the following:

If you need to take your mask off during the day **DO NOT** place it on any surfaces as the outer side of the mask is now contaminated.



When you are not using your mask during the day please place it in your paper bag with the outside of the mask facing your name on the paper bag to keep the inside of the mask clean.

**DO NOT** place your dirty mask in your freezer bag. Remember that under **NO** circumstances should anyone share a mask.



Always remember to wash your hands before putting your mask on, after touching your mask, and after taking your mask off.



At the end of the day please take your mask home in the brown paper bag and wash it. Dispose of the brown paper bag.



Once your mask is clean please place it in the freezer bag so that it is ready to use again at your next shift.

